



Fit Test Adapter Kit #22 For Moldex



OHD Fit Test Adapter | Kit #22



Fig. 1



Fig. 2

INSTRUCTIONS:

1. Be sure that the blue gasket is in place before starting test (Fig. 1)
2. Gently insert the adapters into the mask as you would a cartridge or filter
3. Connect the clear tubes to the two-connector side and the blue tube to the single-connector side (Fig. 2)
4. Perform fit test

CARE:

1. The adapters should be handled with care so as not to create damage that might interfere with testing.
2. Periodically check the metal hose connectors to be sure they are secure.
3. If the diaphragm should leak on the single-port side of the adapter, remove 8 screws on top, carefully line up new diaphragm, screw in the screws loosely. Tighten opposite sides until all are secure.
4. Adapters may be cleaned with soapy water and soft cloth.
5. Adapters can be disinfected by spraying a disinfectant between tests or immersed in a cleaning solution and left out to dry. (The adapters allow outside air to be inhaled, but an employee's exhalation is routed through the exhalation valve nullifying the risk of cross-contamination.)