

Adapter Instructions

Scott 805059-01
For use with Scott AV2000/AV3000

Please follow instructions when using Scott adapter 805059-01 adapter with OHD part 9513-0130 Kit #1 fit test adapter.

The transparent Inhalation valve (1) must be removed from the Scott adapter. This valve is permanently attached, so you must use a needle-nose pliers to grab the diaphragm to remove it.

Please note that the foam-rubber seal on the adapter (2) that creates the actual seal against the mask will eventually wear out. It is recommended that you have a back-up adapter and plan to replace the adapter each year.

